

Fitting *Eyezen+* Lenses

Eyezen+ lenses require the following standard measurements:

- Monocular Distance PD
- Fitting Height (Simply measure from center pupil to the lowest point of the lens to acquire Fitting Height. Recommended minimum fitting height is 15mm)

Eyezen+ lenses with embedded Smart Blue Filter™ feature are also recommended with Crizal® No-Glare treatments for comprehensive eye protection.

The following laser engravings located on the nasal side of each lens will identify the **Eyezen+** design received.

Eyezen+ 1



Eyezen+ 2



Eyezen+ 3



Please visit EyezenPro.com/resources for additional information and materials.



Embedded **Blue Light Protection.**

Eyezen™+

Eyezen™+ LENSES AVAILABILITY

for Eyecare Professionals

Effective MARCH 2016

MATERIAL	PRODUCT	INDEX	COLOR	POWER RANGE	BASE CURVES	MAX CYLINDER
PLASTIC	1.50 Index Clear	1.50		-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
	1.50 Index Essilor Polarized	1.50	Gray, Brown, Gray-Green	-10.00 to +6.00	2.00, 3.25, 4.25, 6.25, 8.25	-4.00
	1.50 Index Xperio UV™ Polarized	1.50	Gray, Brown, Gray-Green	-10.00 to +6.00	2.00, 3.25, 4.25, 6.25, 8.25	-4.00
	1.50 Index Transitions® Signature™ VII	1.50	Gray, Brown, Graphite Green	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
	1.50 Index Transitions® Vantage™	1.50	Gray	-10.00 to +6.00	2.25, 4.25, 5.25, 8.25	-6.00
	1.50 Index Transitions® XTRActive®	1.50	Gray	-10.00 to +6.00	2.25, 4.25, 5.25, 8.25	-6.00
	1.50 Index Transitions® XTRActive®	1.50	Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
AIRWEAR® (POLYCARBONATE)	Clear	1.59		-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Essilor Polarized	1.59	Gray, Brown, Gray-Green	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Xperio UV™ Polarized	1.59	Gray, Brown, Gray-Green	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® Signature™ VII	1.59	Gray, Brown, Graphite Green	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Transitions® XTRActive®	1.59	Gray, Brown	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Airwear® Transitions® Vantage™	1.59	Gray	-10.00 to +8.00	1.50, 3.25, 4.25, 5.50, 7.00, 8.50	-6.00
	Airwear® Colors	1.59	Gray, Brown, Gray-Green	-10.00 to +8.00	2.25, 4.25, 6.50, 7.50, 8.50	-6.00
TREXA™ (TRIVEX)	Clear*	1.53		-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
	Transitions® Signature™ VII*	1.53	Gray, Brown	-10.00 to +6.00	2.00, 3.50, 4.25, 6.25, 7.25	-6.00
THIN & LITE®	1.60 Clear	1.60		-12.00 to +8.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.50	-6.00
	1.60 Transitions® Signature™ VII	1.60	Gray, Brown	-12.00 to +8.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.50	-6.00
	1.67 Clear	1.67		-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	1.67 Essilor Polarized	1.67	Gray, Brown, Gray-Green	-14.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-6.00
	1.67 Xperio UV™ Polarized	1.67	Gray, Brown, Gray-Green	-14.00 to +9.00	1.75, 2.75, 3.75, 5.25, 6.75, 8.00	-6.00
	1.67 Transitions® Signature™ VII	1.67	Gray, Brown, Graphite Green	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	1.67 Transitions® XTRActive®	1.67	Gray, Brown	-14.00 to +9.00	1.75, 2.75, 3.75, 6.00, 6.75, 8.00	-6.00
	1.74 Clear*	1.74		-20.00 to +13.00	0.75, 1.75, 2.75, 3.75, 5.25, 6.50, 6.75, 8.00, 10.00	-6.00
	1.74 Transitions® Signature™ VII*	1.74	Gray	-20.00 to +13.00	0.75, 1.75, 2.75, 3.75, 5.25, 6.50, 8.00, 10.00	-6.00

NOTES: Thin & Lite 1.74 lenses come systematic with choice of Crizal® Previncia™, Crizal Sapphire UV™, Crizal Avancé UV™ or Crizal Alizé UV™
 Please include fitting height when ordering Eyezen+ lenses. Recommended minimum fitting height is 15mm.
 Eyezen+ jobs cannot currently be tinted or treated with lab applied UV.
 * Available Summer 2016.

Eyezen+ Lens Designs.

There are three unique **Eyezen+** designs, and the patient's age will determine which **Eyezen+** lens is right for them. The difference in design is the amount of accommodative relief the patient will receive in the lens.

Eyezen+ 1 —patients age 18 to 34
Eyezen+ 2 —patients age 35 to 44
Eyezen+ 3 —patients age 45 to 50**

**It is recommended that you consider progressive lenses for your patients age 45-50.

For more information on **Eyezen+** lenses, please visit EyezenPro.com

©2016 Essilor of America, Inc. All Rights Reserved. *Essilor* is a registered trademark and *Eyezen* is a trademark of Essilor International. *Transitions* and the *swirl* are registered trademarks and *Transitions Signature* is a trademark of Transitions Optical, Inc. **Photochromic performance is influenced by temperature, UV exposure, and lens material.** *Trivex* is a registered trademark of PPG Industries Ohio, Inc. LEZN000025 SHK ECSB 3/16

